



Ostomy Armor
A Veteran owned business
Comfort. Support. Protection Value: Guaranteed
1 800 308 7637



Ostomy Armor Fit instructions. There is a measuring video on the website.

You will need: Body/Torso size. Foam choice. Color and Model. Buckle choice.

Body measurement:

- The measurement is taken around the torso where the stoma is. Do not guess or use the clothing size.
- Measure right under the stoma and pull the tape measure tight.
- If the customer thinks they will lose or gain a lot of weight, include that information.
- E.G. after chemotherapy many people regain the weight they lost from ill health.
- Being more active (with the Armor) allows people to return to exercise.
- Ostomy Armor has about 2-3" of room to tighten or loosen.
- Indicate the side of the stoma: left/right

Foam pad sizes:

- Foam is available in Thin/.5", Standard/.75" or Deep/1"
- A very small stoma such as a urostomy, or one with little protrusion, will only require the thin foam.
- Under normal clothing it's hard to see the Armor. Under spandex/super tight clothing, the profile of the product will be seen.
- Most stomas that protrude up to one inch will work well with the Standard foam.
- Stomas that protrude more than an inch ALL THE TIME will feel more comfortable with the Deep foam. The Deep foam will be more prominent under clothing. (Only chose if the stoma NEVER retracts.)
- If the stoma retracts at all to under an inch, chose the Standard foam for comfort and a slimmer profile.
- Cut-out sizes (the back of the cut-out shape in the foam.) Adult sizes are all 50mm. Childrens' Armor cutout is 45mm. The cutout allows waste to drain into the bag.

Buckle choice.

- Each model comes complete with a strong buckle as standard issue. Each model has the option of: a) super-strong Cobra Buckle or b) an easy open buckle for hands with limited dexterity. The website has a buckle video.

Dual Armor:

- Dual Armor choices include foam thickness and feeding tube/catheter option. Each side is adjustable to fit the configuration of each stoma/catheter/tube. Also suitable for hernias.
- Buckle choices are also available on the Dual: Cobra buckles and the Body Point buckle.
- Contact us for help measuring. We will need photos of the abdomen with a tape measure in the frame.

Measuring for a hernia:

- Estimate how far the hernia can be pulled/pushed in and include that information. Large hernias over 3" high may be best suited to the Large Hernia product.

Many custom options and alterations are available. Please contact us.

Please feel free to contact us with any questions you may have about our products.

Gillian Wilks Gwilks@ostomyarmor.com or info@ostomyarmor.com

WWW.OSTOMYARMOR.COM

